208*109*60mm

S

RÉKAVICE LATEKSOVE

rozmiar / size
7 (S) - 9 (S)

3520 002 100 07

LATEX GLOVES

100 kusů / kusov
sztuk / pieces

- ✓ Nesterilní / Nesterilné / Niesterylne / Non-sterile
- ✓ Přírodní latex / Přírodný latex / Latex naturalny / Natural latex
- ✓ Jednorázové / Jednorazové / Jednorazowe / Disposable
- ✓ Vyšetřovací / Vyšetřovacie / Do badań lekarskich / Examination
- ✓ Lehce pudřené / Zľahka pudrované / Lekko pudrowane / Slightly powdered
- ✓ Obouruké / Obojručné / Oburęczne / Ambidextrous
- ✓ Jen pro minimální rizika / Len pre minimálne riziká / Tylko dla ryzyka minimalnego / Minimum safety risk only

BERT

3520 002 100 07

EN 420:2003+A1:2009
EN 455-1/2/3

Dovceca / Dovceca / Importer / Importer
CANIS SAFETY s.r.o. Chudobská 2307/69, Hlohovec, 198 00, Praha 9,
Czech Republic
www.canis.cz

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EN

User informations
Complies with the requirements of Regulation (EU) 2016/425. They are in compliance with the Regulation (EU) 2017/745 concerning medical devices (class I).

Instructions for use:
The gloves are made of high-quality latex. The gloves are designed for minimum safety risk. The gloves provide protection of hands against non-aggressive dirt - weak detergents with easily rectifiable impact (the gloves provide protection against diluted detergents etc.). The gloves are intended for medical examination; they are a non-sterile medical item, class I, EN 455-1/2/3. Put the gloves on dry and clean hands.

Please note:
The material of gloves may be damaged by kerosene, mineral oils, fats or organic solvents. The gloves are made of natural latex, which may cause an allergic reaction, including anaphylactic shock. If skin irritation or allergic reaction do not use gloves and contact your doctor immediately. Do not use damaged gloves. Do not use the gloves for handling sharp objects, flames or hot or heated objects. The gloves do not protect against chemical, biological, mechanical, or other risks! A little powder is applied on the gloves. The gloves do not provide full coverage of wrists. Storing: They should be kept in a dry and well ventilated room. Excessive humidity, temperature, or intensive light may influence quality of gloves. The supplier doesn't provide warranty for such damage. **Expiration life and transport:** The shelf life is set at 5 years, provided the storage and transport conditions are observed. The month/year of production and expiration is marked on the box. Transport the product in the original packaging. The gloves should be disposed of in accordance with the Waste act. The Declaration of Conformity can be found here: www.canis.cz; for individual products, in the bar "Downloads".

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EN

User informations
Complies with the requirements of Regulation (EU) 2016/425. They are in compliance with the Regulation (EU) 2017/745 concerning medical devices (class I).
Instructions for use:
The gloves are made of high-quality latex. The gloves are designed for minimum safety risk. The gloves provide protection of hands against non-aggressive dirt - weak detergents with easily rectifiable impact (the gloves provide protection against diluted detergents etc.). The gloves are intended for medical examination; they are a non-sterile medical item, class I, EN 455-1/2/3. Put the gloves on dry and clean hands.
Please note:
The material of gloves may be damaged by kerosene, mineral oils, fats or organic solvents. The gloves are made of natural latex, which may cause an allergic reaction, including anaphylactic shock. If skin irritation or allergic reaction do not use gloves and contact your doctor immediately. Do not use damaged gloves. Do not use the gloves for handling sharp objects, flames or hot or heated objects. The gloves do not protect against chemical, biological, mechanical, or other risks! A little powder is applied on the gloves. The gloves do not provide full coverage of wrists. Storing: They should be kept in a dry and well ventilated room. Excessive humidity, temperature, or intensive light may influence quality of gloves. The supplier doesn't provide warranty for such damage. **Expiration life and transport:** The shelf life is set at 5 years, provided the storage and transport conditions are observed. The month/year of production and expiration is marked on the box. Transport the product in the original packaging. The gloves should be disposed of in accordance with the Waste act. The Declaration of Conformity can be found here: www.canis.cz; for individual products, in the bar "Downloads".

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